



Instructions for Use

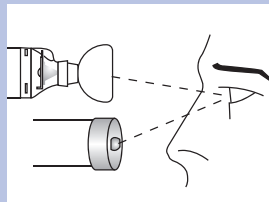


READ INSTRUCTIONS PRIOR TO USE


Prior to use, carefully examine the product for proper assembly, foreign objects and damaged or missing parts. Remove any foreign objects prior to use. The product should be replaced **IMMEDIATELY** if there are any damaged or missing parts.

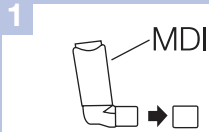


Remove cap from Metered Dose Inhaler (MDI). Before use, ensure the instructions supplied with the MDI have been read.

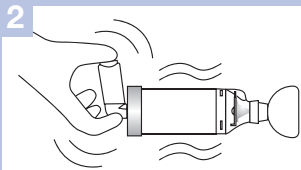


Insert the MDI into the **Backpiece** of the chamber. While holding the **AeroKat*** Feline Aerosol Chamber (FAC) and MDI firmly together, shake the unit briskly 4 to 5 times immediately before each use.

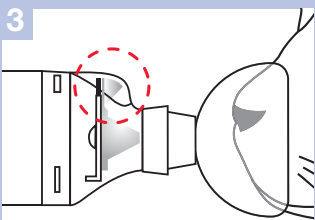


Gently apply mask to face and ensure an effective seal. As the cat inhales, the **Flow-Vu*** Inspiratory Flow Indicator (IFI) should flutter or move toward the cat. If your cat experiences anxiety, speak to your veterinarian for advice.

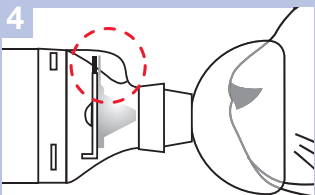

The **Flow-Vu*** IFI will return to the vertical position when the cat stops inhalation or exhales.

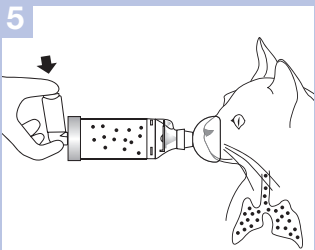


As the cat inhales, depress the MDI and hold mask in place for 5 to 6 breaths. Maintain an effective seal and use the **Flow-Vu*** IFI to assist in breath counting. Ensure the veterinarian's dosage instructions are followed and that only one puff of medication is delivered at a time.

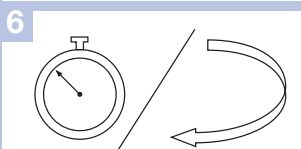


Follow instructions supplied with the MDI regarding the amount of time to wait before repeating instructions 2 – 5 as prescribed.


Cleaning Instructions

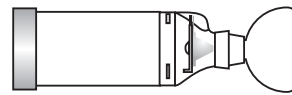

The **AeroKat*** FAC with **Flow-Vu*** IFI should be cleaned before first use and then weekly.

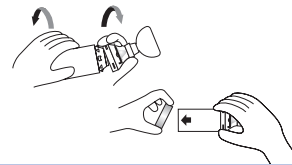


Remove the **Backpiece**, mask adapter and mask only. **DO NOT** attempt to remove any other parts.

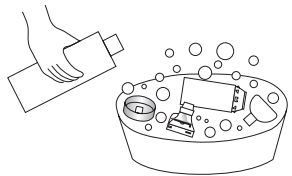


Soak all parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently.




Rinse parts in clean water.

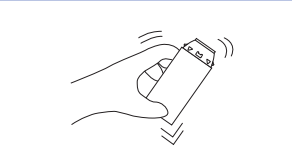

Shake out excess water from the parts and allow to air dry in a vertical position.

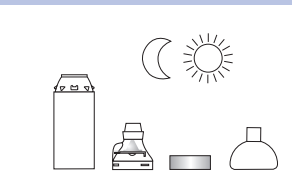

Ensure parts are dry before reassembly. Do not rub dry.

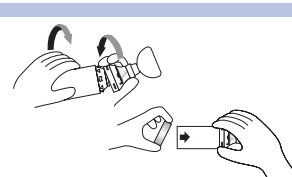

Replace **Backpiece**, mask adapter and mask when unit is completely dry and ready for use.

Intended Use

For use with a Metered Dose Inhaler as prescribed by your veterinarian.

Cautions

- Use only under the direction of a veterinarian and read all instructions before use.
- Do not leave **AeroKat*** FAC unattended with children.
- Do not disassemble the product beyond what is recommended in the Cleaning Instructions or damage may result.
- The **AeroKat*** FAC is a medical device. To ensure proper performance it should be cleaned according to these instructions. It should **NOT** be cleaned in a dishwasher.
- If unsure how to use this product, talk to your veterinarian.

Notes

- Read and follow the instructions provided with your MDI.
- This product contains no latex.
- If you have questions about the performance or usability of this product, please contact your veterinarian.
- FAC – Feline Aerosol Chamber
- For instructions in alternate languages, visit our website (www.aerokat.com)



Manufactured by:

Trudell Medical International*

725 Third Street,
London, Ontario, Canada N5V 5G4
Fax: +1(519) 455-7858 • e-mail: aerokat@trudellmed.com
www.aerokat.com

AeroKat[®]



Feline Aerosol Chamber

